


PERFEKTAN® ACTIVE

Dosing table

Solution	1,0 %	2,0 %	3,0 %
4 Liter	40 g	80 g	120 g
	1 X 	2 X 	3 X 
	2 X 	4 X 	6 X 
8 Liter	80 g	160 g	240 g
	2 X 	4 X 	6 X 
	4 X 	8 X 	12 X 
12 Liter	120 g	240 g	360 g
	3 X 	6 X 	9 X 
	6 X 	12 X 	18 X 
20 Liter	200 g	400 g	600 g
	5 X 	10 X 	15 X 
	10 X 	20 X 	30 X 
28 Liter	280 g	560 g	840 g
	7 X 	14 X 	21 X 
	14 X 	28 X 	42 X 
40 Liter	400 g	800 g	1200 g
	10 X 	20 X 	30 X 
	20 X 	40 X 	60 X 

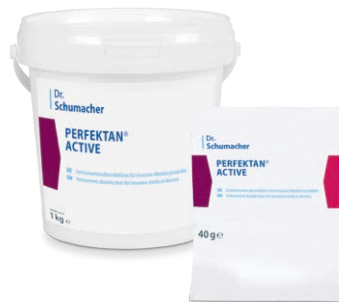

Preparing the working solution

1. Read off the concentration and quantity in the dosing table.
2. Pour in the required amount of water of suitable quality (e.g., deionized water) below 30 °C.
3. Add the specified amount of PERFEKTAN ACTIVE to the water.
4. Stir several times and dissolve the powder within 15 minutes.
5. The solution can be used after 15 minutes.

Notes on the working solution

- The working solution must be renewed immediately when visibly soiled, but at least every working day (max. 12 hours). PERFEKTAN ACTIVE must not be mixed with other process chemicals.
- Ensure adequate ventilation when using the working solution

Legend:

1 X  = 20 g 1 X  = 40 g

Composition

Active ingredient: Peracetic acid (in-situ) > 850 ppm
(1 % dilution), hydrogen peroxide

Single Unit	Unit	Content	REF
sachet	100	40 g	00-155-0004
bucket	6	1 kg	00-155-010