



Reprocessing of DESCO WIPES and ECO WIPES non-woven wipes dispenser by ULTRASOL OXY® reprocessing wipes




Each time the DESCO WIPES and ECO WIPES non-woven wipe dispenser is refilled, the system must undergo a preventive cleaning and disinfection. Along with possible chemical-thermal and thermal procedures, manual preparation using ULTRASOL OXY® reprocessing wipes is possible. ULTRASOL OXY® reprocessing wipes are provided with every refill package. It is recommended to inspect the non-woven wipes dispenser before reprocessing. Replace cover / dispenser when defects show. For filling of non-woven wipes dispensers and handling disinfectants observe the applicable rules for personnel protection and wear suitable protective equipment.

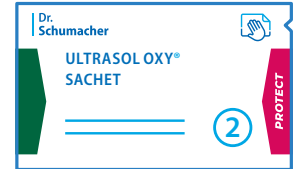

CLEANING ①

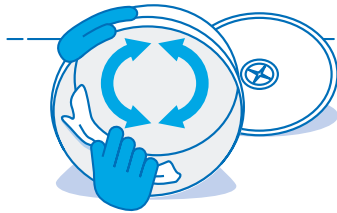

Thoroughly clean the insides and edge of the new or emptied dispenser with the cleaning wipe.

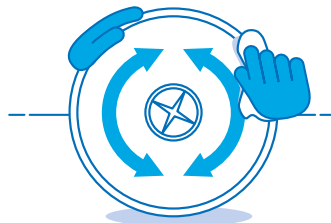

Thoroughly clean the insides of the cover, the edges of the cover, inside surfaces and the edges of the dispenser opening.




Thoroughly clean the outside of the cover and the outside surfaces of the dispenser.




DESINFECTION ②



Thoroughly wipe the insides and edge of the new or emptied dispenser with the disinfection wipe.

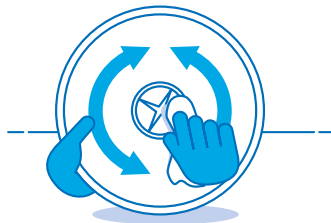

Thoroughly wipe the insides of the cover, the edges of the cover, inside surfaces and the edges of the dispenser opening.

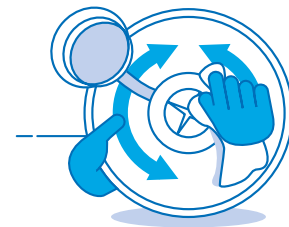

*** Observe contact times**

Thoroughly wipe the outside of the cover and the outside surfaces of the dispenser.

