

Instructions for Use: Cleaning and Disinfection ULTRASOL OXY®



Sporicidal surface disinfection and cleaning of invasive medical devices with and without mechanical action.



PROTECT

Where can I use ULTRASOL OXY®?


Stethoscopes



Incubators



Surfaces in
Food Processing

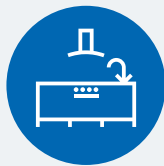

Ultrasound Probes



Dialysis Machines




Ventilation Masks



How do I properly use ULTRASOL OXY®?

- Put on gloves² and other appropriate PPE.
- Slowly pour the disinfectant over the roll of a nonwoven wipes dispenser in a spiral motion. Wait for wipes to fully saturate.
- Pull the first wipe out of the roll and guide it through the dispensing system.
- Remove visible soiling. Wipe the surface completely and keep moist for the entire contact time.

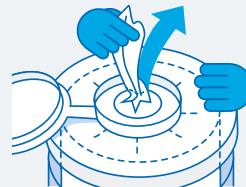

What spectrum of efficacy does ULTRASOL OXY® offer and what are the exposure times?¹

Sporicidal EN 17126 EN 17846	Sporicidal <i>C. difficile</i> EN 17126 EN 17846	Tuberculocidal and Mycobactericidal EN 14348	Virucidal EN 14476	Fungicidal EN 13624 EN 16615	Bactericidal / yeastcidal VAH / EN 16615
15 min	5 min	5 min	5 min	5 min	1 min

¹ For a more detailed overview of efficacies and exposure times, please reference the product information sheet.
² Follow the manufacturer's instructions and the guidelines in the ULTRASOL OXY safety data sheet.

Use disinfectants safely. Always read the label and product information before use.