

Instructions for Use: Cleaning and Disinfection ULTRASOL OXY® WIPES and XL WIPES



Sporicidal surface disinfection and cleaning of invasive medical devices with and without mechanical action.



PROTECT

Where can I use ULTRASOL OXY® WIPES and XL WIPES?


Stethoscopes



Incubators



Surfaces in
Food Processing

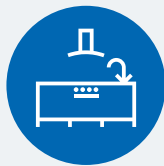

Ultrasound Probes



Dialysis Machines





Ventilation Masks



How do I properly use ULTRASOL OXY® WIPES and XL WIPES?

- Put on gloves² and other appropriate PPE.
- Lift resealable lid and remove a single wipe.
- Remove visible soiling. Wipe the surface completely and keep moist for the entire contact time.
- Close the lid after use to prevent the wipes from drying out quickly.

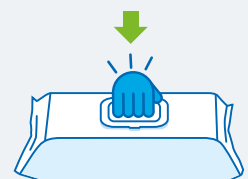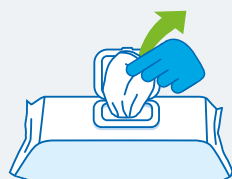

What spectrum of efficacy does ULTRASOL OXY® WIPES and XL WIPES offer and what are the exposure times?¹

Sporicidal EN 17126 EN 17846	Sporicidal C. difficile EN 17126 EN 17846	Tuberculocidal and Mycobactericidal EN 14348	Virucidal EN 14476	Fungicidal EN 13624 EN 16615	Bactericidal / yeastocidal VAH / EN 16615
15 min	5 min	5 min	5 min	5 min	5 min

¹ For a more detailed overview of efficacies and exposure times, please reference the product information sheet.
Use disinfectants safely. Always read the label and product information before use.