

Instructions for Use: Cleaning and Disinfection DESCOCEPT SENSITIVE WIPES with Biodegradable Wipes



Ready-to-use, alcohol based wipes for cleaning and disinfection of surfaces and medical devices.



No gloves required!¹



PRESERVE

Where can I use DESCOCEPT SENSITIVE WIPES?

Stethoscopes



Monitors



Countertops and
Work Surfaces



Blood Pressure
Cuffs



Call Buttons and
Control Panels





Door Handles



How do I properly use DESCOCEPT SENSITIVE WIPES?

- Lift resealable lid and remove a single wipe.
- Remove visible soiling. Wipe the surface completely and keep moist for the entire contact time.
- Close the lid after use to prevent the wipes from drying out quickly.

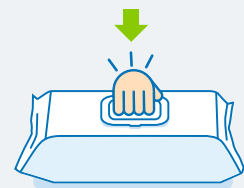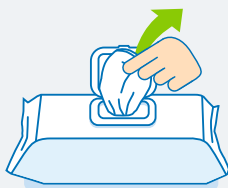

What spectrum of efficacy does DESCOCEPT SENSITIVE WIPES offer and what are the exposure times?²

Norovirus EN 14476	Effective against all enveloped viruses EN 14476	Tuberculocidal EN 14348	Bactericidal / yeastcidal VAH / EN 16615	Adenovirus EN 14476	Limited virucidal EN 14476
ab 30 s	1 min	1 min	2 min	3 min	3 min

¹ Provided that infection control and occupational safety regulations allow. Dermatologically tested.

² For a more detailed overview of efficacies and exposure times, please reference the product information sheet. Biodegradability certified according to the European standard EN 13432.

Use disinfectants safely. Always read the label and product information before use.